




## Getting Started

Visit our website and Select Providers Portal or visit <https://viewimages.peninsulaimaging.com>

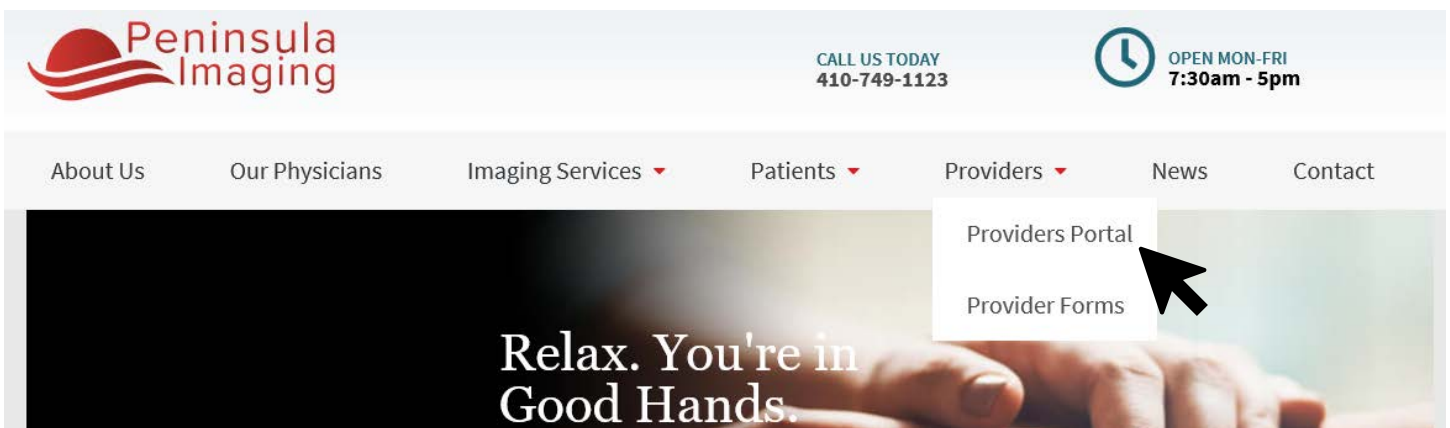

Log in to view your study exams and reports

Log In  
User ID:   
Password:   
Preferred Language:   
English (United States)   
Register

Enter your credentials and select a preferred language.

Using the Studies tab, enter patient search information and press Enter or click Search


Studies | Inbounds | Outbounds | Conference Folders  
Last Name:  First Name:  Patient ID:  Date of Birth:  Modality:   
Accession No.:  Institution:  Study Performed:  Ref. Physician:  Issuer of PID:   
Search Preset:   
My Search

Additional Search Features

**Save** allows you to save the current search criteria as a **Search Preset** or **My Search**.  
**Clear** resets all fields except for the Study Performed date.  
Select any **Search Preset** from the drop-down and click **Search**.

Double-click a patient study to open it

Study Date	Modality	Gender	Patient Name	Description	Patient ID	F-Images	Last Name	Patient DOB	Body Part	Accession
07-Aug-2009 8:40:13 AM	OT/NOCLAR	F	MULTIPLEPRIORS, JANE	MR BRKETS BLAT	A00137	2879	MULTIPLEPRIORS	18-Sep-1948		A00137-3
01-Dec-2013 12:00:00 AM	PT/CT	F	MULTIPLEPRIORS, JANE	RET CT SKULL/THIGH	A00137	880	MULTIPLEPRIORS	18-Sep-1948		A00137-22
07-Apr-2012 12:00:00 AM	CT	F	MULTIPLEPRIORS, JANE	CTA HEAD W/ CONTRAST	A00137	775	MULTIPLEPRIORS	18-Sep-1948		A00137-19
12-Apr-2012 11:10:28 AM	DOCT	F	MULTIPLEPRIORS, JANE	CT RIGHT UPPER EXTREMITY W/ CONTRAST	A00137	527	MULTIPLEPRIORS	18-Sep-1948		A00137-9
08-Apr-2013 4:00:35 PM	DOCT	F	MULTIPLEPRIORS, JANE	CT SHOULDER W/ CONTRAST	A00137	489	MULTIPLEPRIORS	18-Sep-1948		A00137-18

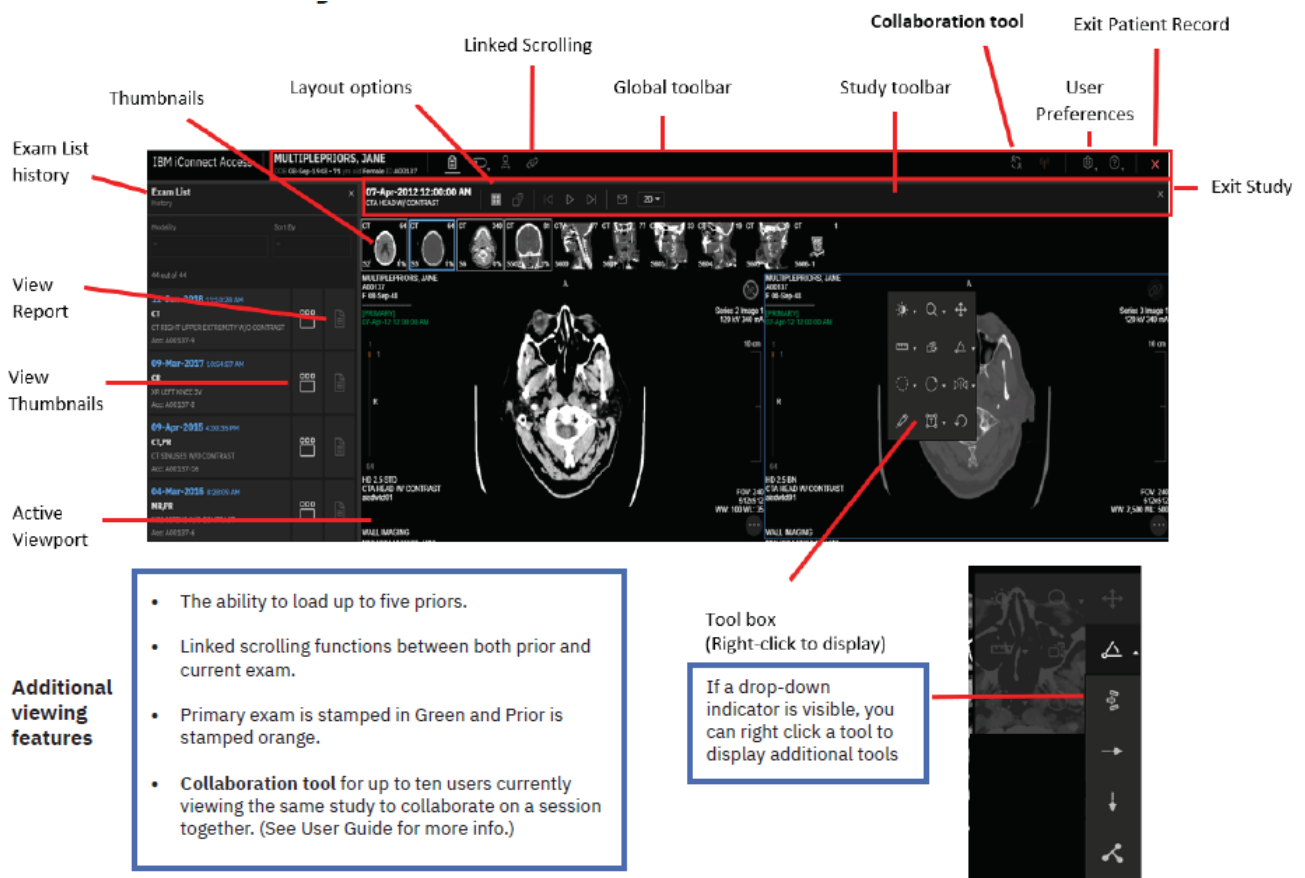

Labels in the screenshot include: Exam List history, View Report, View Thumbnails, Active Viewport, Thumbnails, Layout options, Linked Scrolling, Global toolbar, Study toolbar, Collaboration tool, Exit Patient Record, User Preferences, and Exit Study.

**Additional viewing features**

- The ability to load up to five priors.
- Linked scrolling functions between both prior and current exam.
- Primary exam is stamped in Green and Prior is stamped orange.
- **Collaboration tool** for up to ten users currently viewing the same study to collaborate on a session together. (See User Guide for more info.)

**Tool box (Right-click to display)**

If a drop-down indicator is visible, you can right click a tool to display additional tools

For iConnect assistance please contact [techsupport@peninsulaimaging.com](mailto:techsupport@peninsulaimaging.com) or call 443-736-4830